

ARJOHUNTLEIGH

GETINGE GROUP

MAXI SLIDE

VERSATILE REPOSITIONING SHEETS



...with people in mind

MAKING EVERYDAY TASKS EASIER AND SAFER

Carers spend a large part of their working day transferring or repositioning patients. *Maxi Slide* helps carers to carry out these frequent tasks in an easier and safer way, without the stress and strain associated with manual handling.

Using *Maxi Slide* is a fast and convenient solution for many handling situations including common bedside activities such as moving, turning or sitting up the patient. Routines involving lateral transfers and patient repositioning can be performed safely and efficiently.


Maxi Slide offers a unique range of simple patient handling aids that can improve the quality of care and create a safer working environment for carers. There are many methods you can select when using sliding material and just a few of them are shown here. The use of any method should follow a suitable risk assessment.



Turning in bed



Solution: two *Maxi Slide* Sheets

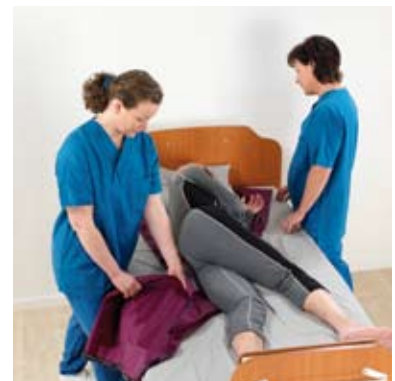

Moving up the bed



Solution: two *Maxi Slide* Sheets and a *Maxi Tube*






Lateral transfer from bed to Concerto



Solution: *Maxi Slide Sheet*, straps and a *Maxi Transfer*



Moving out of a confined space on the floor

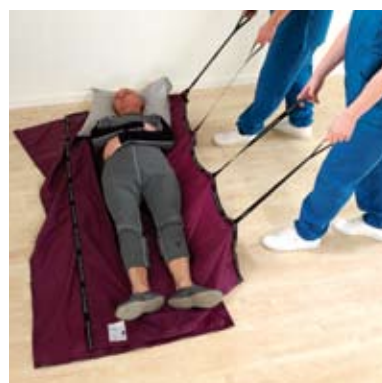

Solution: two *Maxi Slide Sheets* and four pull straps




Fitting a sling using Maxi Slide



Solution: two *Maxi Slide Sheets*


PRODUCT DETAILS

Maxi Slide Sheets

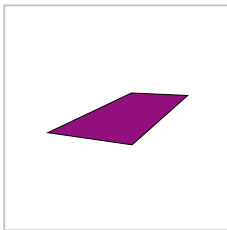


MAXI SLIDE SHEET

- 730 mm (33") x 1730 mm (68")
- 730 mm (33") x 2000 mm (78 ¾")

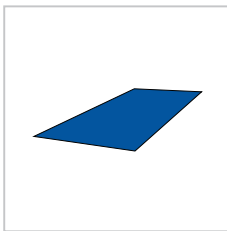

MAXI SLIDE SHEET XL

- 1095 mm (43") x 1730 mm (68")
- 1095 mm (43") x 2000 mm (78 ¾")

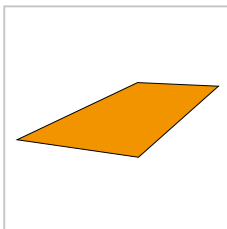

MAXI SLIDE SHEET XXL

- 1460 mm (57 ½") x 1730 mm (68")
- 1460 mm (57 ½") x 2000 mm (78 ¾")

Mobility Gallery™

Suitable for Carl, Doris and Emma



C Carl, who sits in a wheelchair and has little capacity to support himself

D Doris, who has no capacity to support herself

E Emma, who is almost completely bedridden and totally dependant


Please contact ArjoHuntleigh for further information on the Mobility Gallery™.

Maxi Tube - Maxi Roll - Maxi Mini - Maxi Transfer




MAXI TUBE

Length 585 mm (23") ø 433 mm (17")

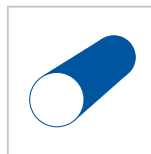


MAXI TRANSFER

Length 1960 mm (77") ø 433 mm (17")

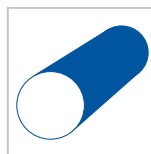

MAXI ROLL

Length 2100 mm (82½") ø 900 mm (35 ½")

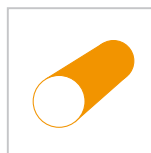

MAXI MINI

Length 900 mm (35 ½") ø 1100 mm (43 ½")



PULL STRAPS

Length 660 mm (26")
Set of eight.

Maxi Slide In-service DVD

To assist your staff further with some other techniques using *Maxi Slides*, an in-service DVD is available covering all the "Mobility Gallery™" from Albert through to Emma.

Only ArjoHuntleigh designed parts, which are designed specifically for the purpose, should be used on the equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice.

® and ™ are trademarks of the ArjoHuntleigh group of companies.
© ArjoHuntleigh, 2010

Note: Maxi Slide has not been designed to, and therefore must not be used to lift patients.

Warning: Ensure that sliding sheets are never dropped or left on the floor, as they are extremely slippery and accidents could occur.

ARJOHUNTLEIGH

GETINGE GROUP

www.ArjoHuntleigh.com

Arjo International AB

PO Box 61

SE-241 21 Eslöv

SWEDEN

Phone: +46 413 645 00

Fax: +46 413 645 63

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.